Cleaning and Sterilizing: 1. Bfree bottles must be used with original Bfree teats only. 2. For hygiene reasons always wash and sterilize the teats before first use. 3. Immediately after each use, wash the teat inside and out in hot soapy water, being careful not to damage the hole. Do not use the anti-bubble tube cleaning brush (provided with the Bfree bottle) as this may tear the teat. 4. Do not turn the teat inside out for cleaning. 5. Teats may be washed in the dishwasher on the top rack. 6. When sterilizing, make sure to follow the manufacturer's sterilizing instructions. Alternatively you can sterilize the teat by boiling it in water for 4-5 minutes. 7. We recommend that you replace teats every 6 to 8 weeks for hygiene purposes as well as wear and tear.

The following warnings are for your child's safety and health:

- * Inspect the teat before and after each use.
- * Replace worn, damaged or punctured teats.
- * Keep teats in a dry covered container. Do not leave in direct sunlight.
- * Feeding teats should not be used as teethers.
- * Always use Bfree products under adult supervision.
- * Continued and prolonged bottle feeding may cause tooth decay.
- * Always check food temperature before feeding.

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The drop shapes molded on the outside of the teat determine the flow rate, and allow you to choose the speed that is most suitable for your baby.

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